Valsure™ Enzymatic Cleaner

Reorder Number: **1C5205** Contents: **18.93 Liters (5 Gal.)**

Valsure Enzymatic Cleaner cleans surgical instruments and other medical devices. This is designed for manual, ultrasonic and automated washer-disinfecter applications. Consult the manufacturer's recommended reprocessing instructions to determine any additional procedures necessary to clean their particular medical device. Medical devices should be sterilized or disinfected by appropriate means in a separate subsequent step.

Directions: Manual/Ultrasonic Application: Fill sink or basin with warm water to the appropriate level to fully immerse surgical instruments. Dilute chemistry 1/2 - 1 fl. oz. per gallon (4 - 8 mL per Liter) of warm water, depending on soil load. Activity increases as the water temperature increases. Soak for a minimum of 2 - 5 minutes. Cleaning times may be longer with dried on proteinaceous materials. If lower temperatures are used, a longer cleaning time may be necessary. Do not exceed 140°F (60°C). After cleaning, all surfaces should be thoroughly rinsed with warm water. Solutions should be discarded when visibly soiled.

Automated Washer-Disinfecter Application: Automatically feed at 1/4 - 1 fl. oz. per gallon (2 - 8 mL per Liter), depending on soil load. This is a complete product. Do not add other chemicals, such as bleach or detergents. Added chemicals may destroy the effectiveness of this product.

STORAGE AND DISPOSAL: Store in a cool place below 90°F (32°C). Thoroughly rinse empty container and deposit in waste receptacle for removal.

DANGER

CONTAINS: Citric acid (CAS# 77-92-9), ethanolamine (CAS# 141-43-5). Causes skin irritation. Causes serious eye irritation. May cause allergy or asthma symptoms or breathing difficulties if inhaled. Avoid breathing mist, vapours, spray. Wash hands thoroughly after handling. Wear protective gloves, protective clothing, eye protection. **IF ON SKIN:** Wash with plenty of water. **IF INHALED:** Remove person to fresh air and keep comfortable for breathing. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. For hospital and professional use only. Not for home use. See Safety Data Sheet for additional information.

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STERIS Corporation
Product Made in U.S.A.